


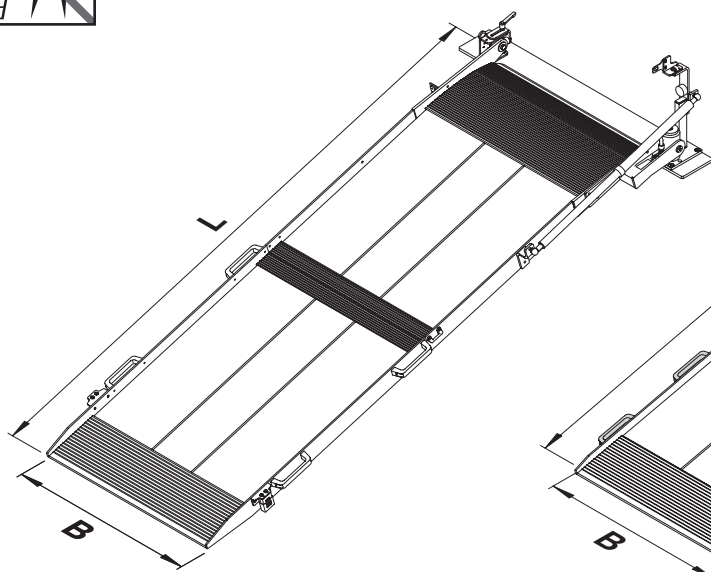
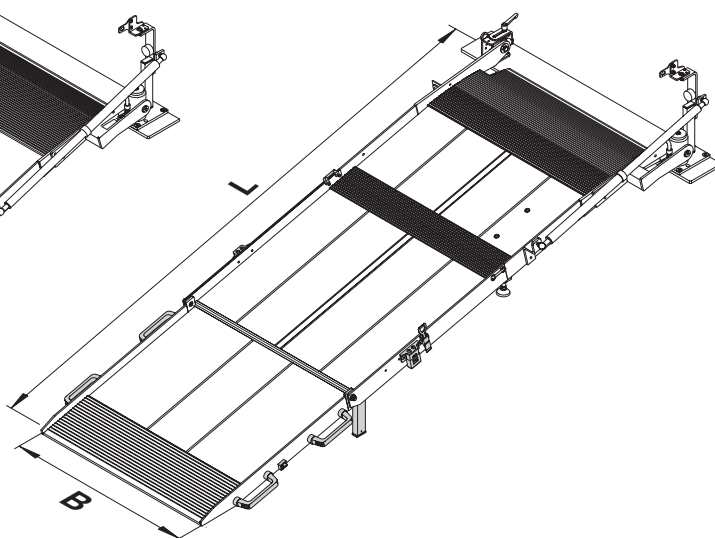
RRD, RRD-Z, RLK-D, RLK-Z-D






1 RRD, RLK-D







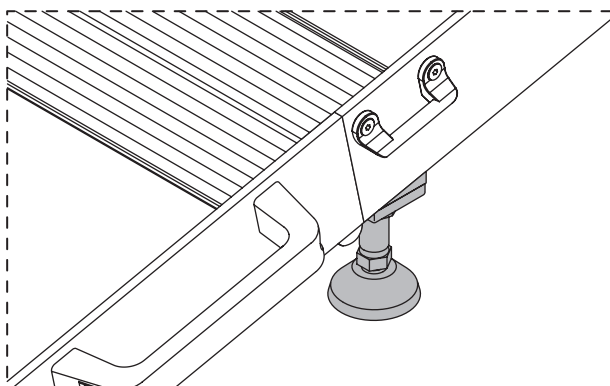
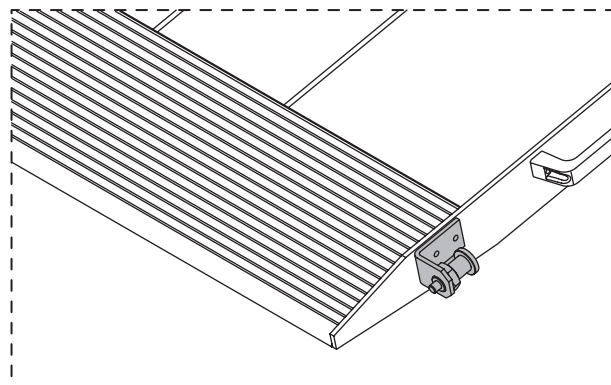
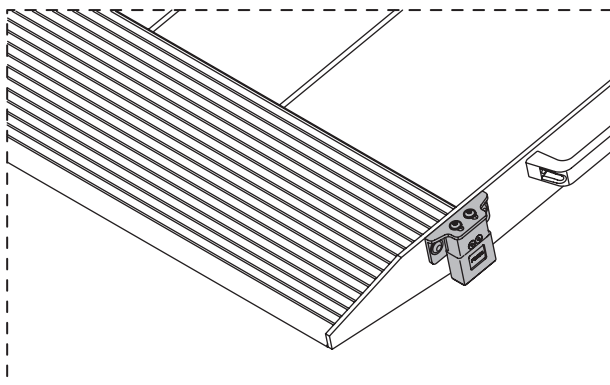
2 RRD-Z, RLK-Z-D

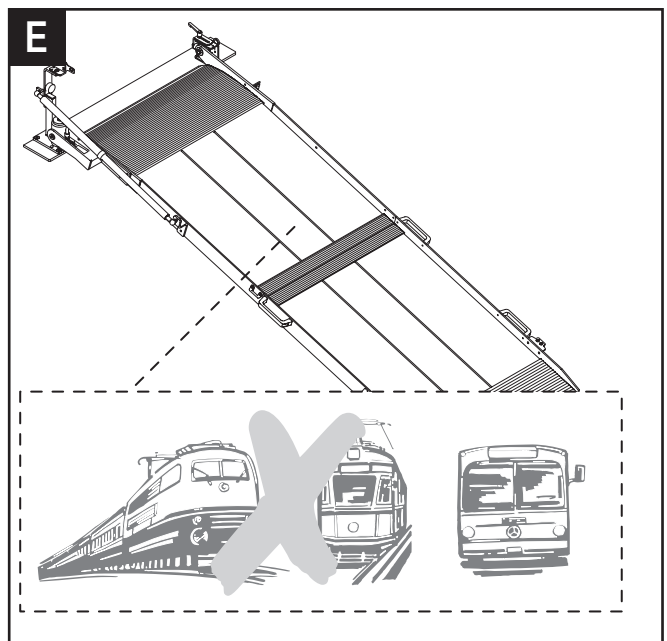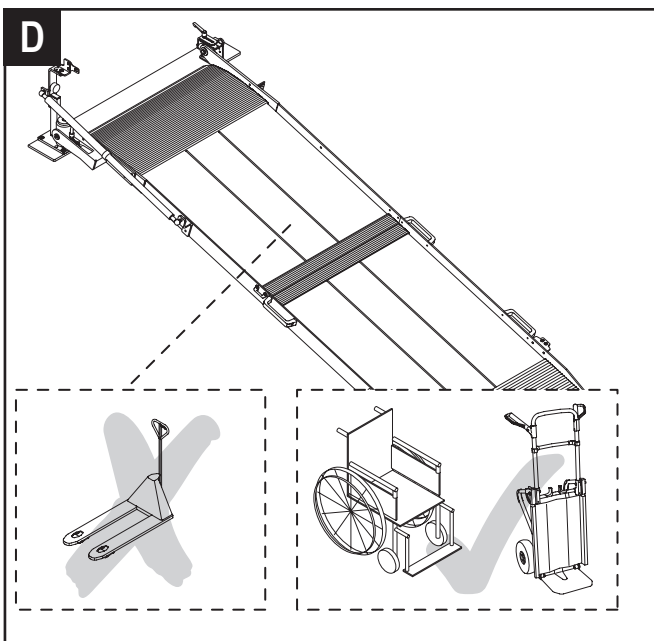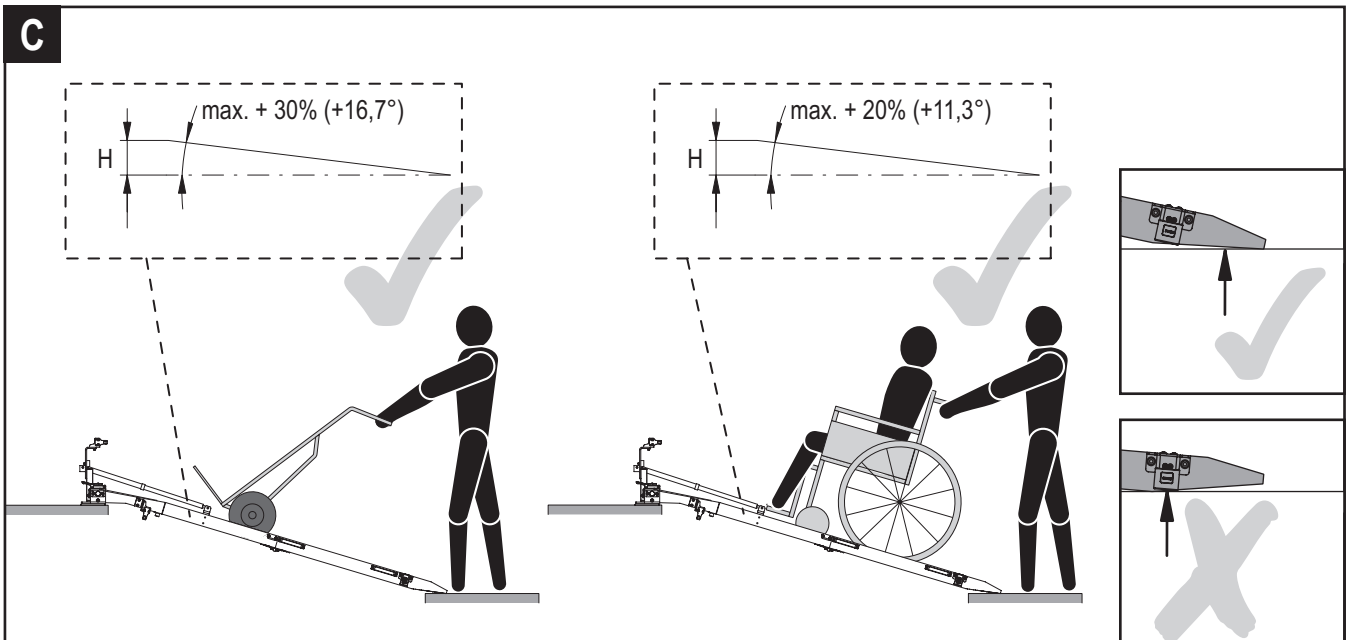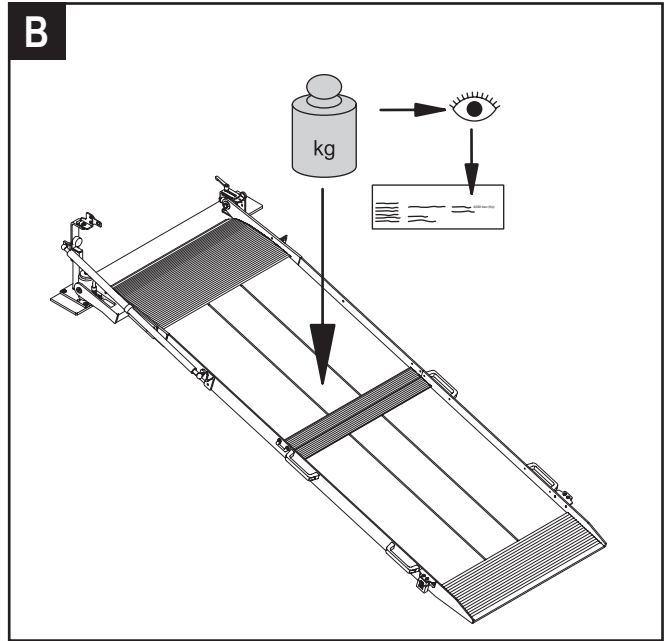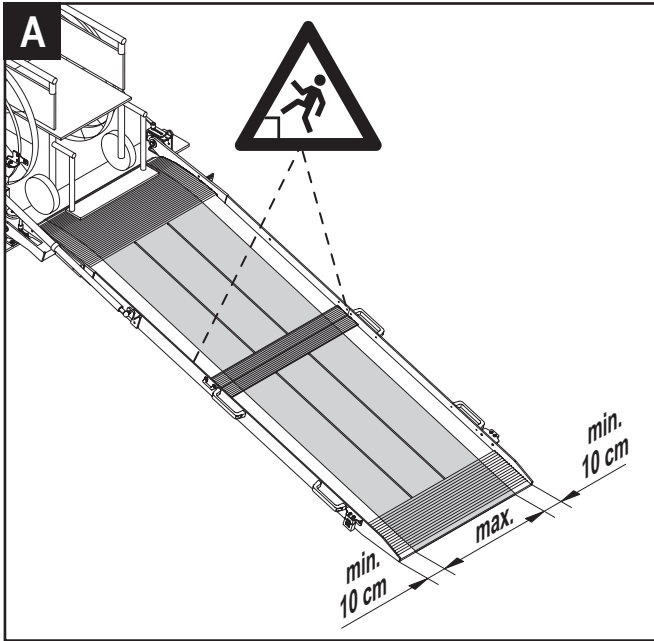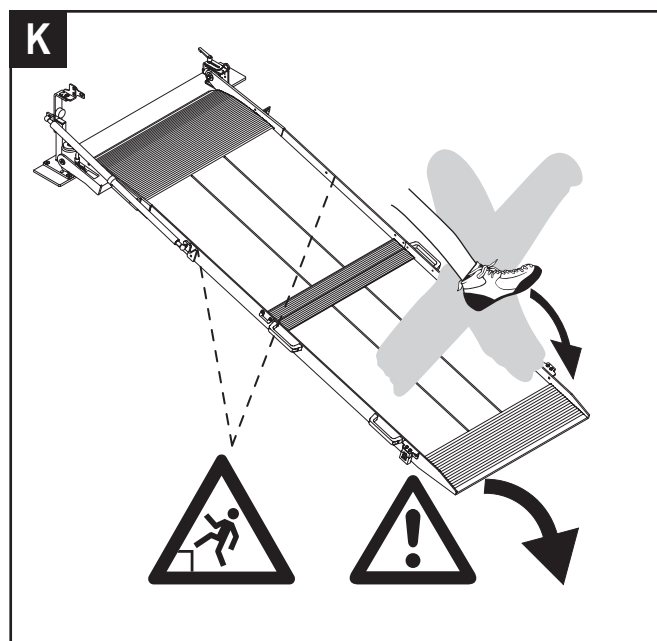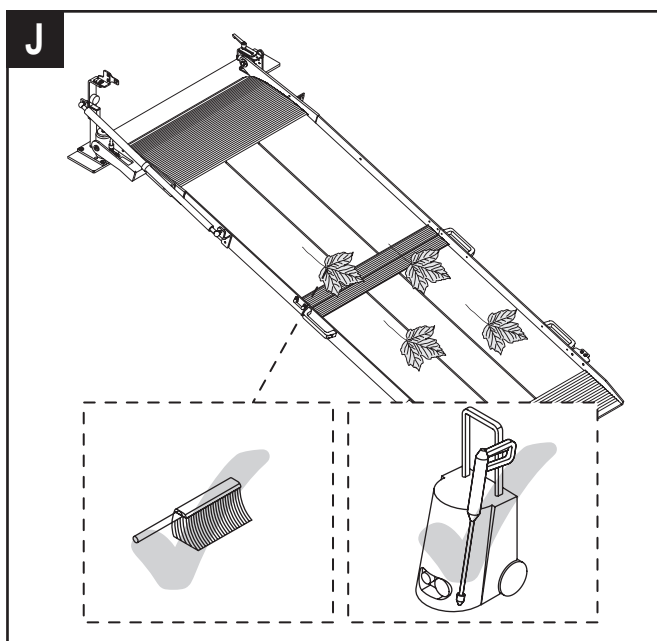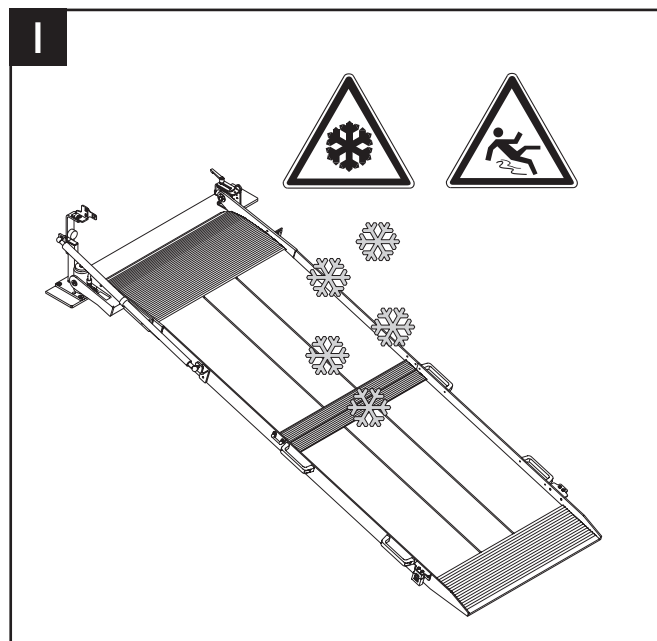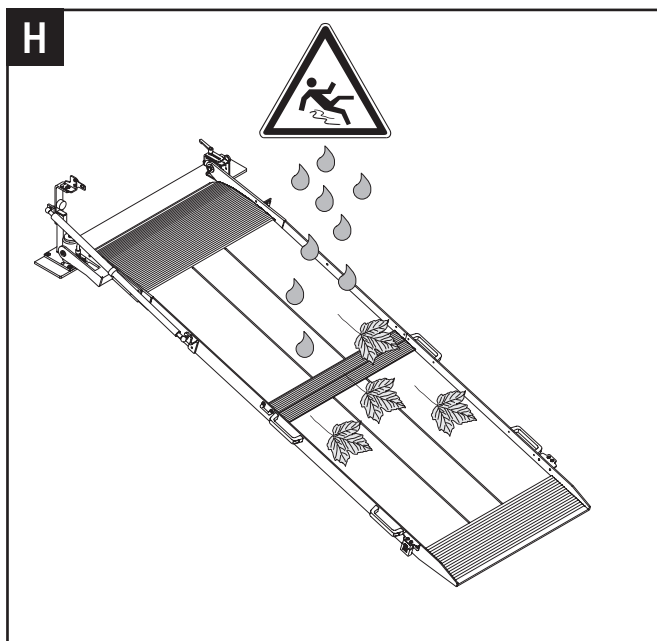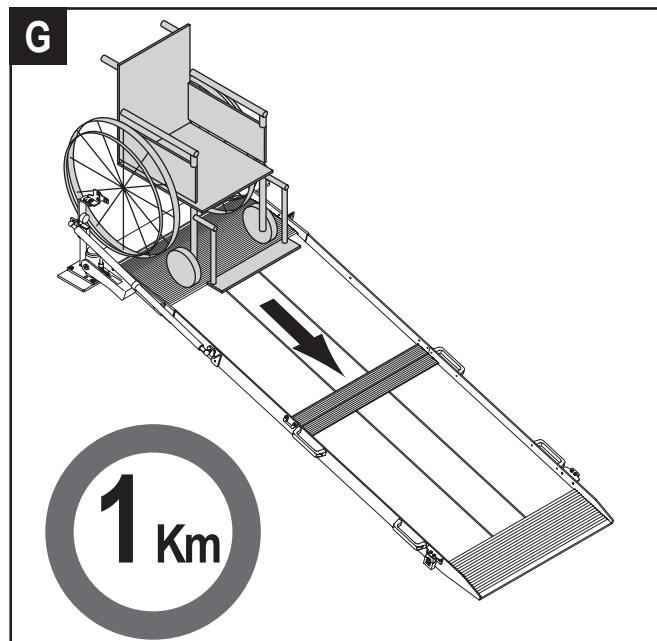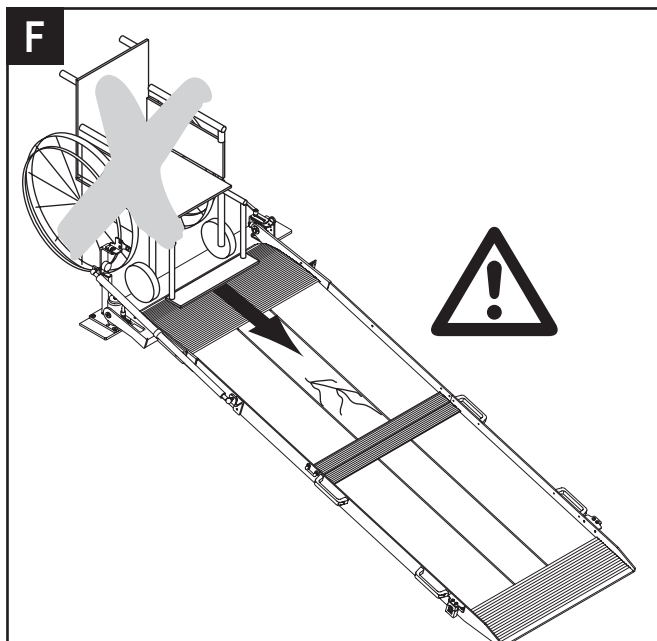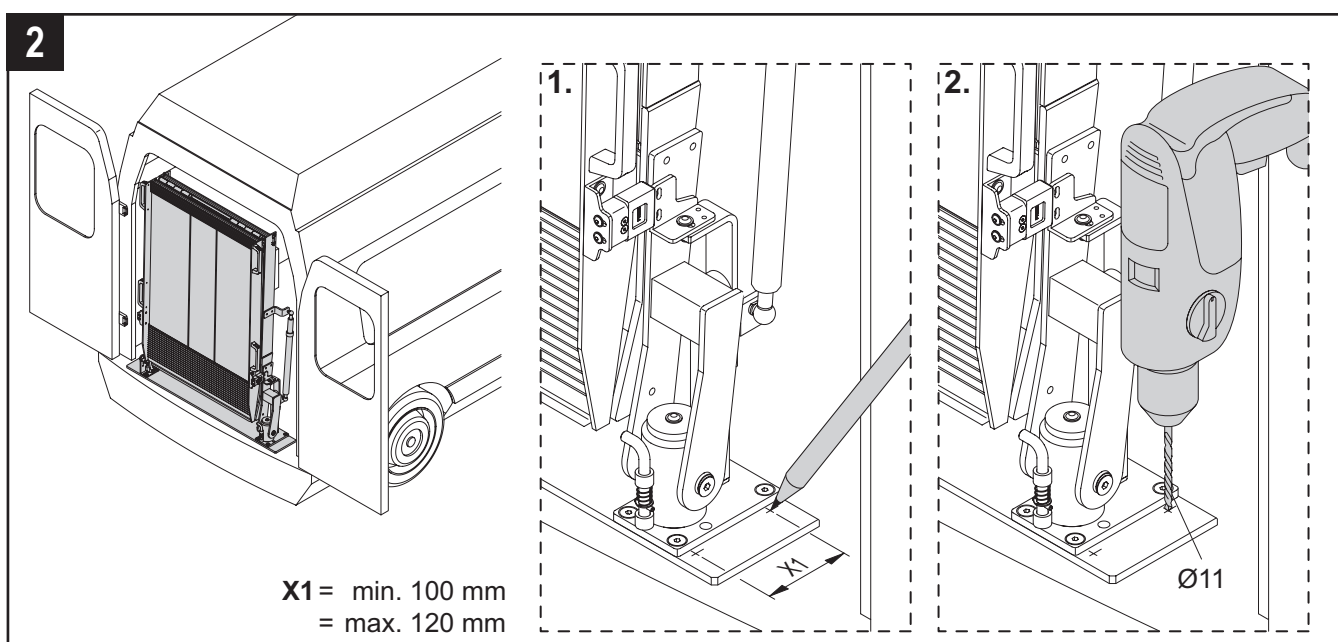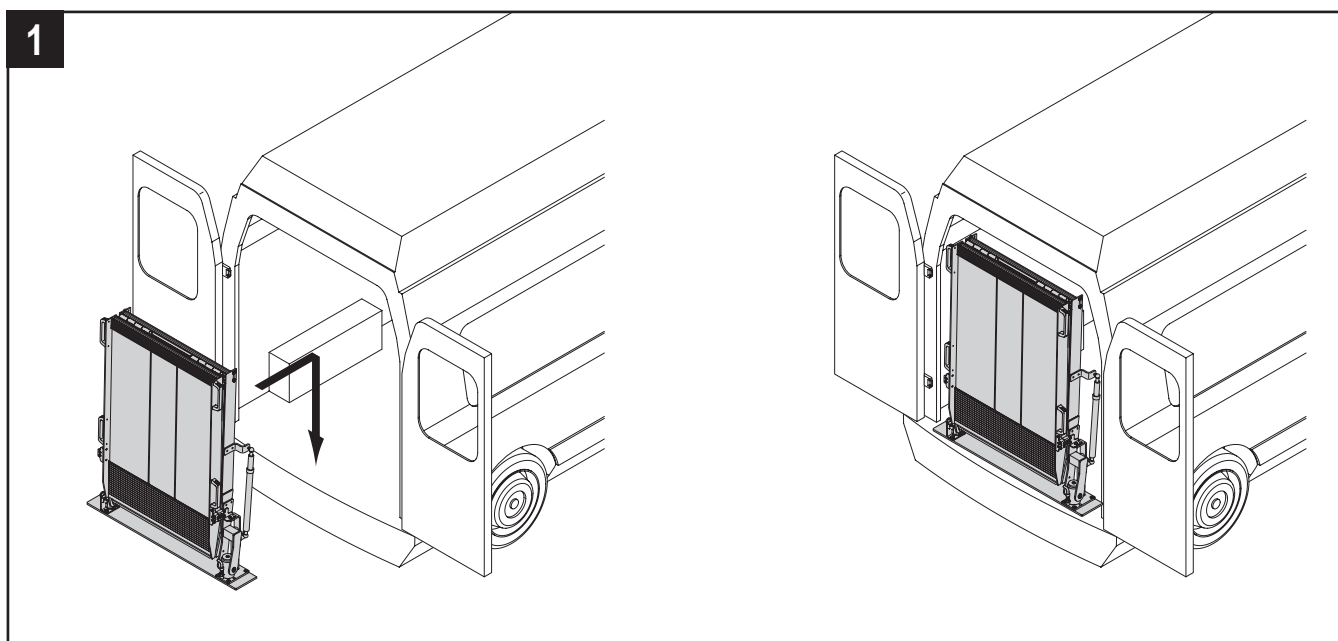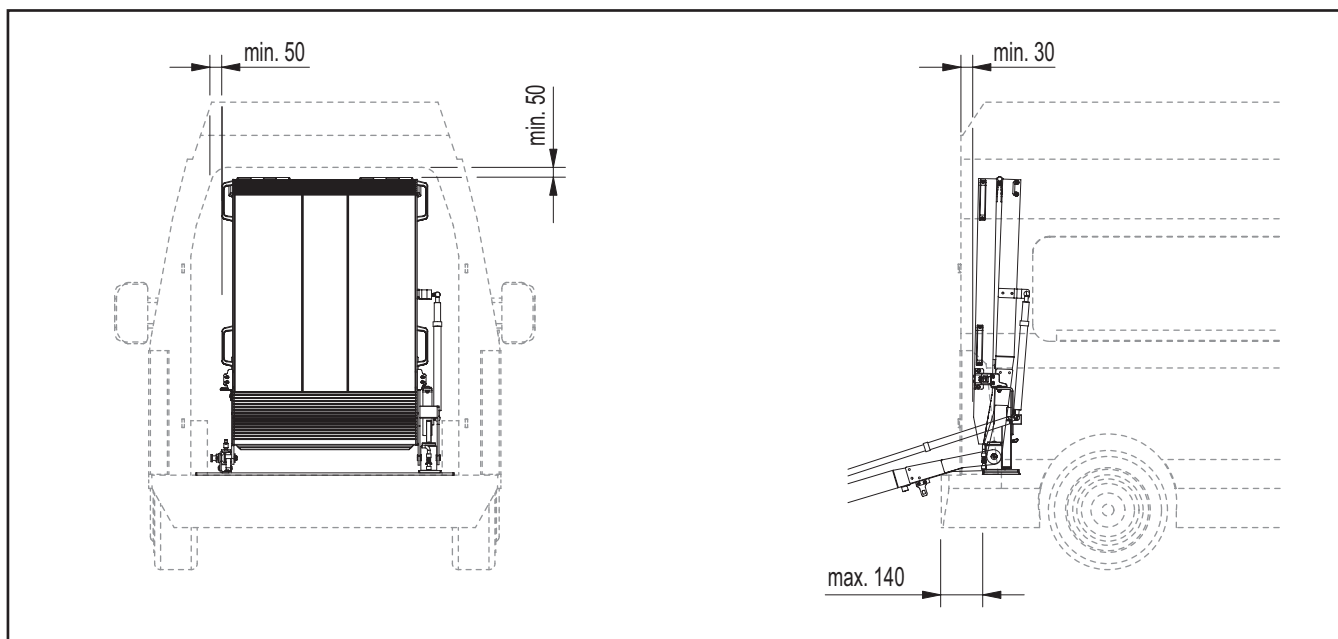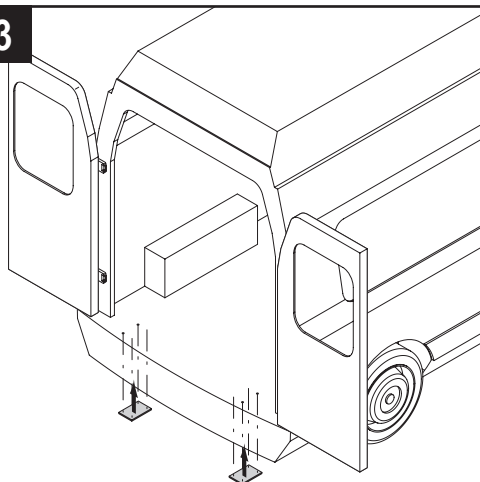
Bedienungsanleitung - Návod k obsluze - Betjeningsvejledning - Instrucciones de uso
- Mode d'emploi - User manual - Kezelési útmutató - Istruzioni per l'uso - Bruksanvis-
ning - Bedieningshandleiding - Manual de operação - Instrukcja obsługi - Инструкция по
эксплуатации - Bruksanvisning - Käyttöohje - návod na obsluhu - Navodila za uporabo -
Kullanım kılavuzu - Εγχειρίδιο οδηγιών













**3**

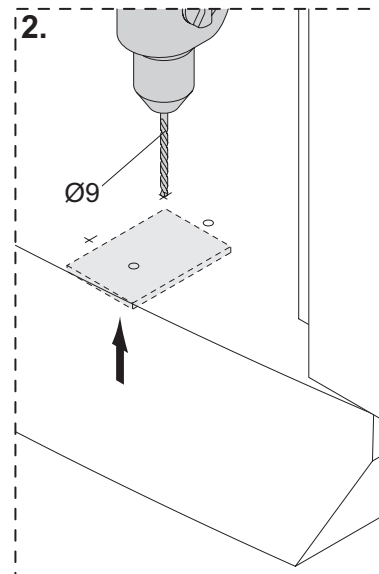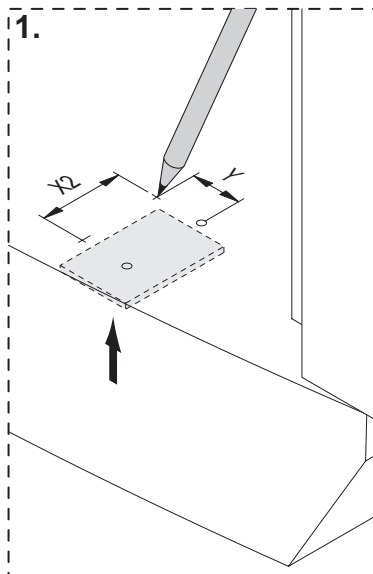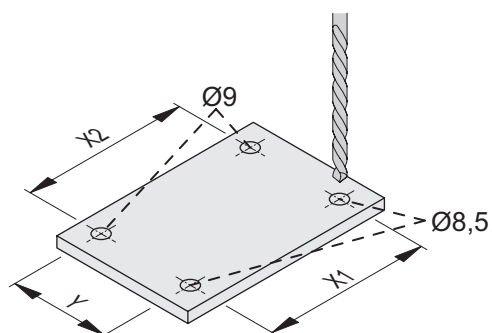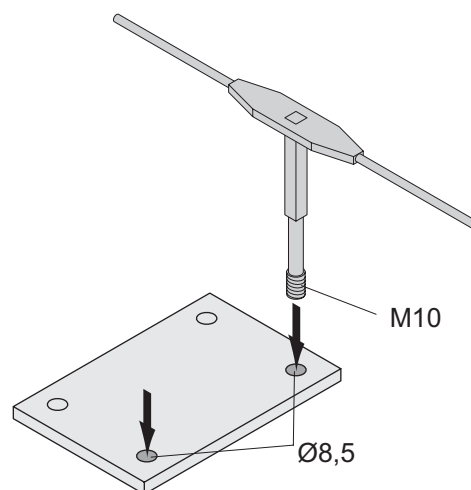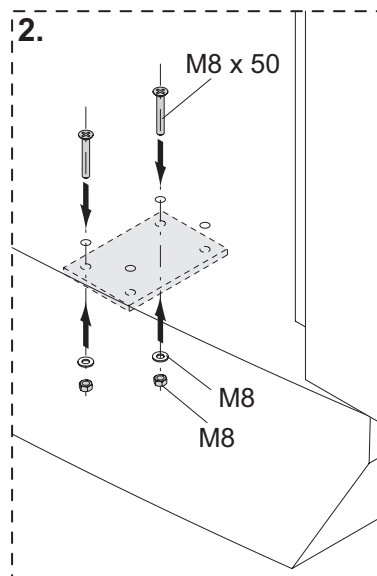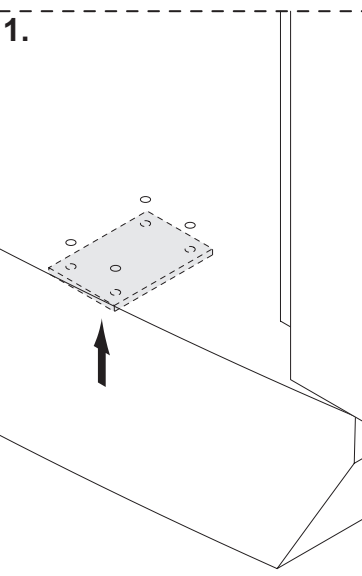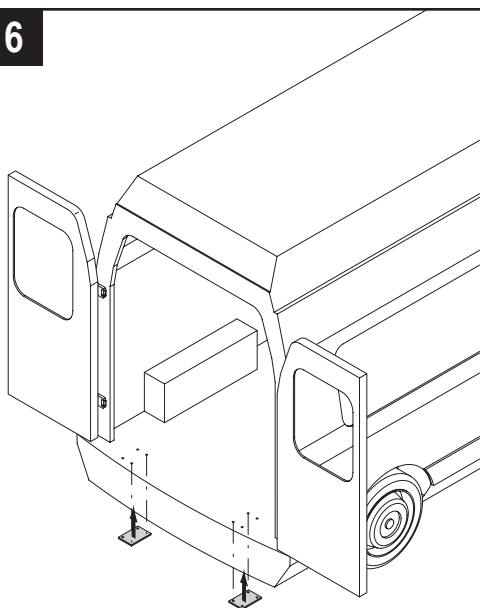X2 = min. 70 mm **Y** = min. 40 mm
 = max. 120 mm = max. 80 mm

**4****5****6**

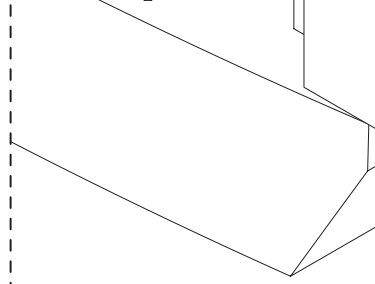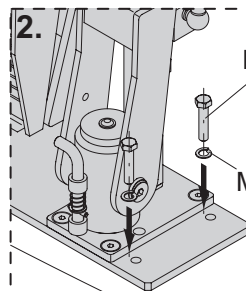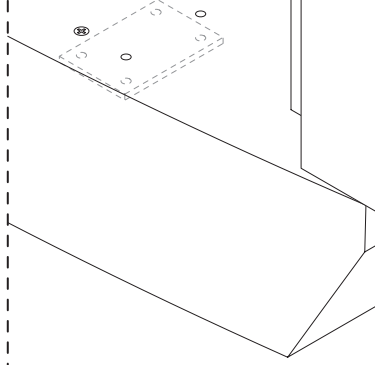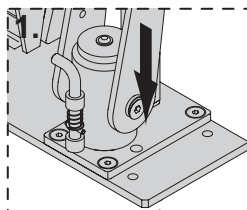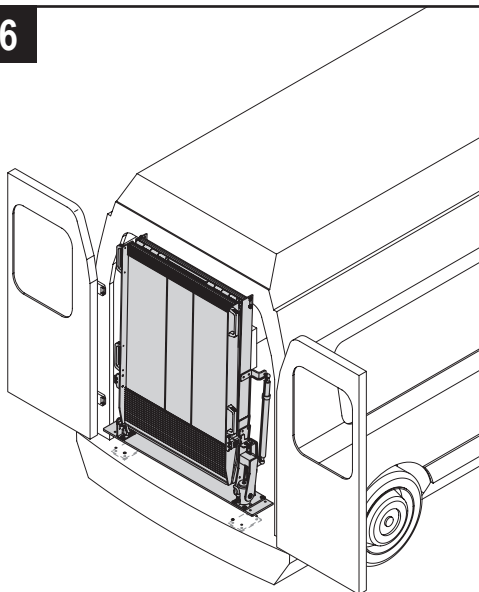6



2.




M10 x 45

M10

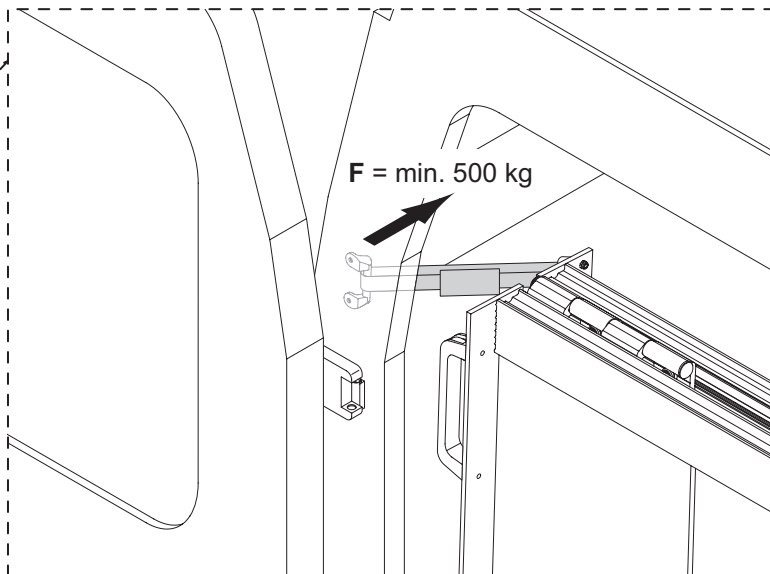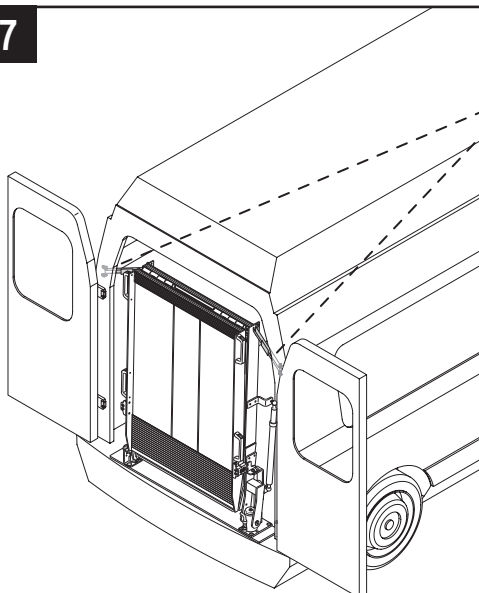
7


F = min. 500 kg






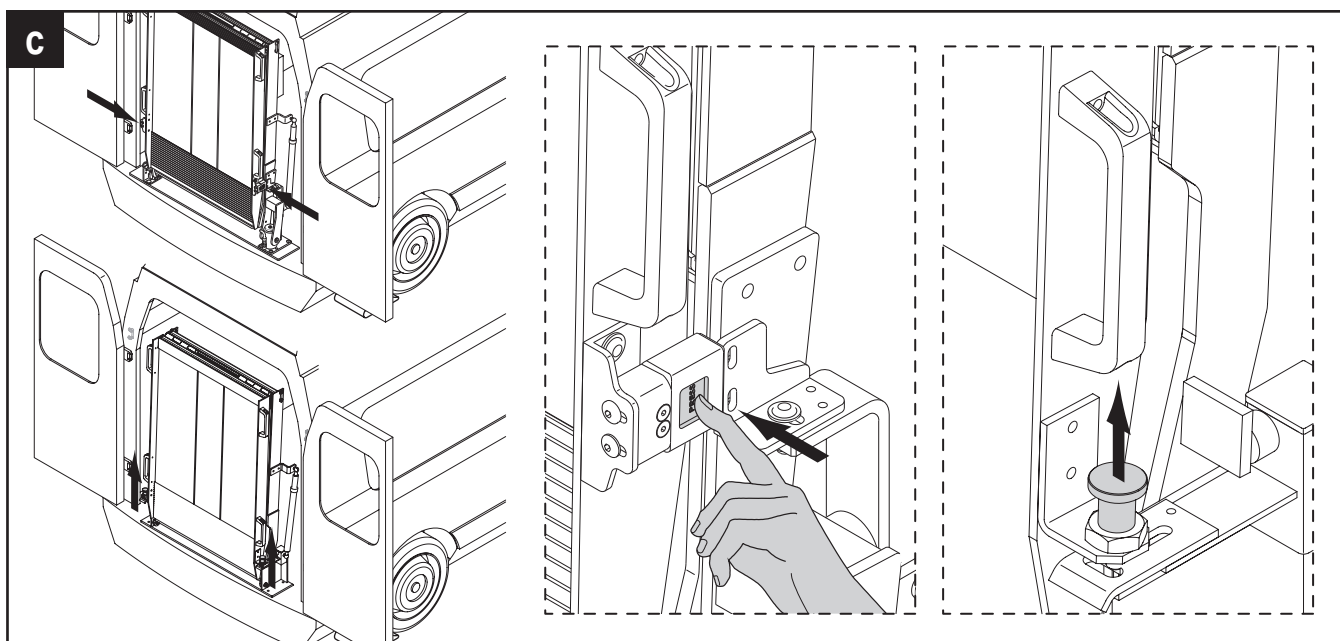
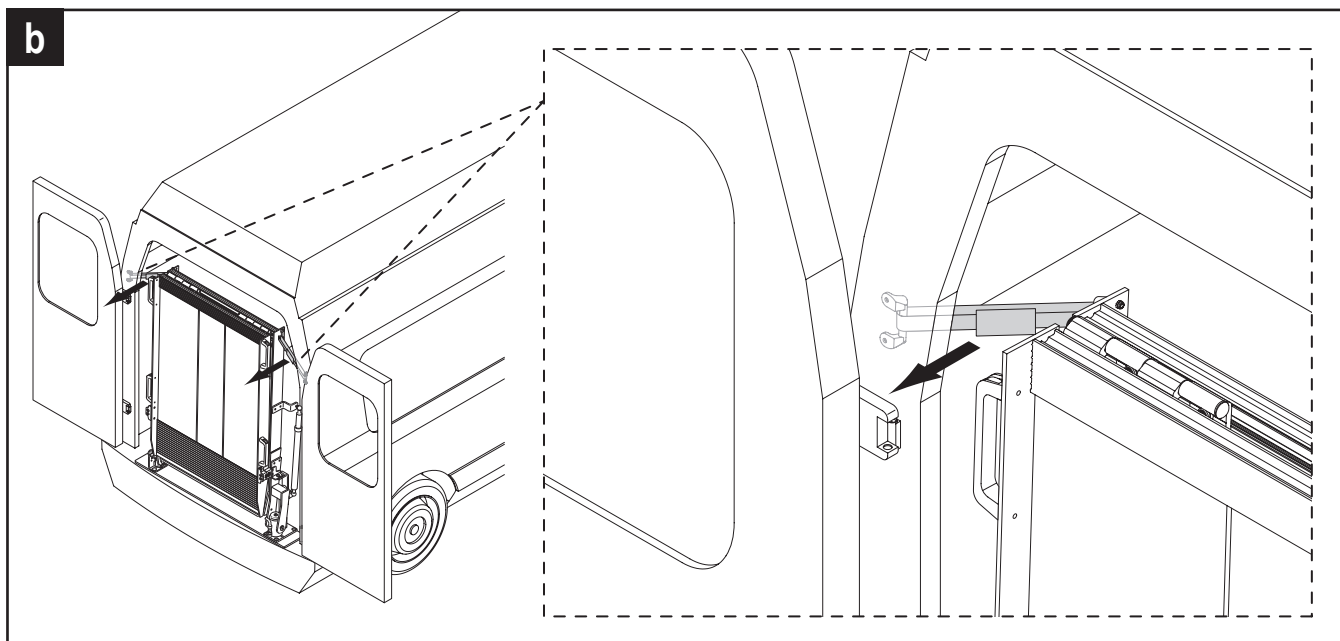
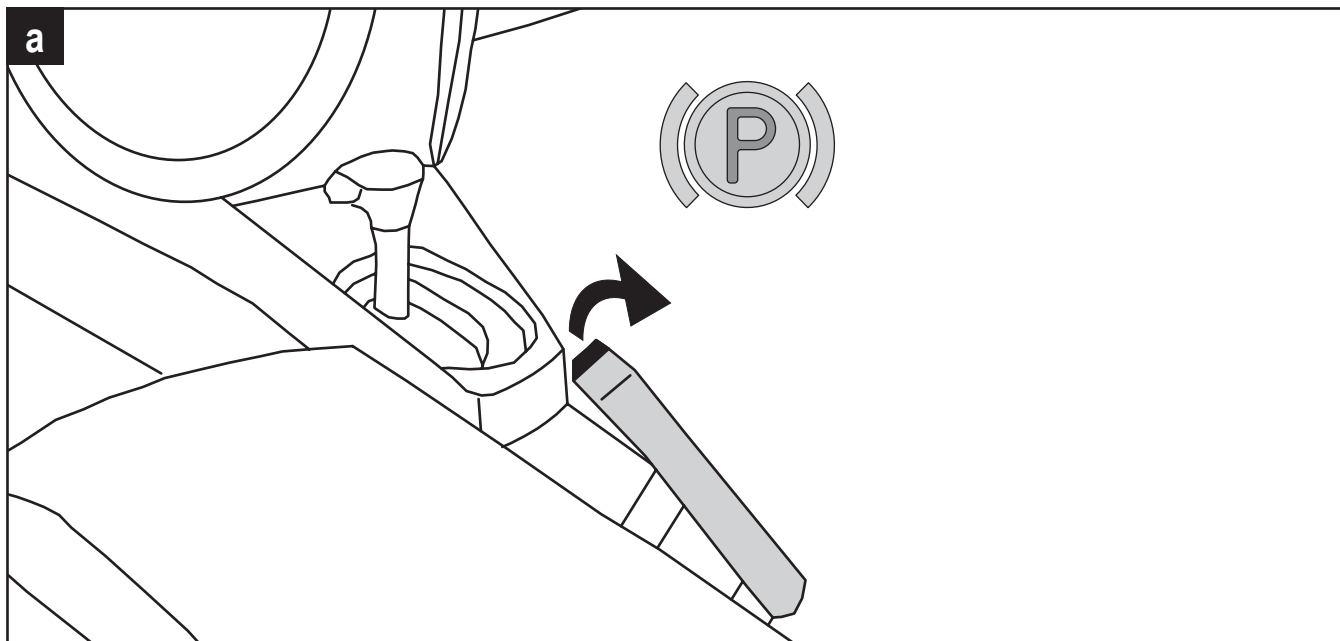
d | **RRD, RLK-D**









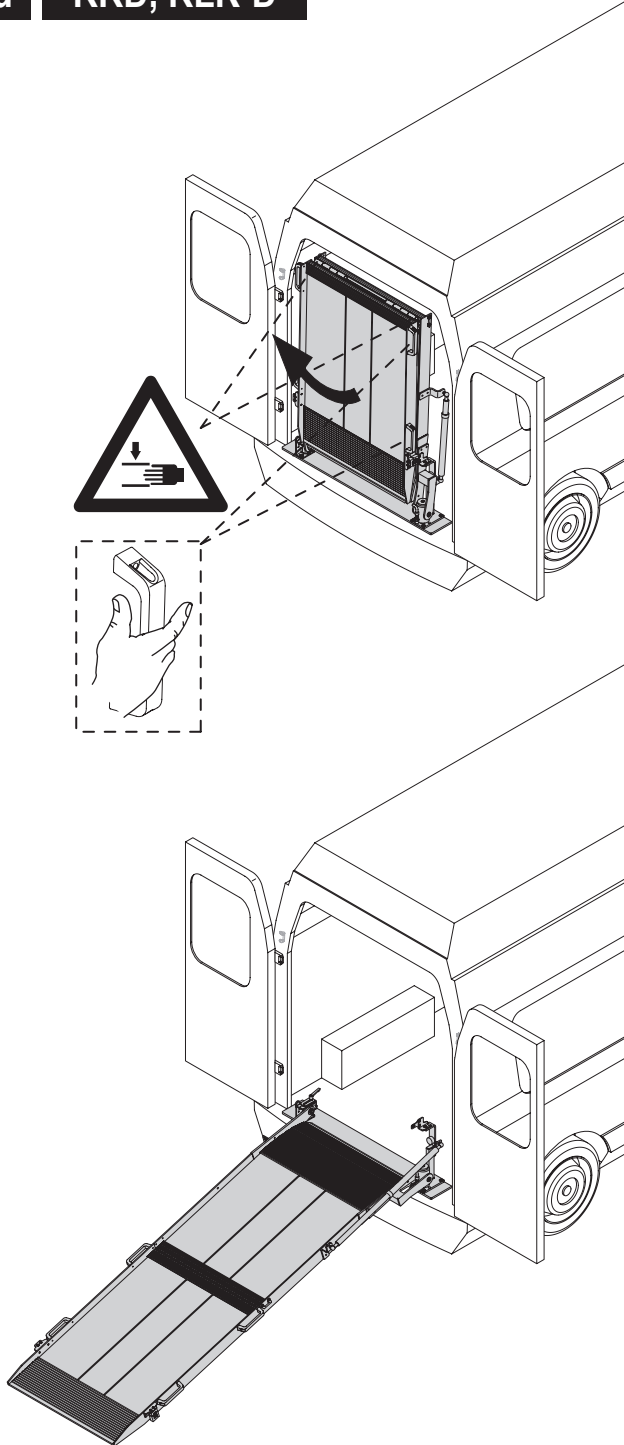
d | **RRD-Z, RLK-Z-D**

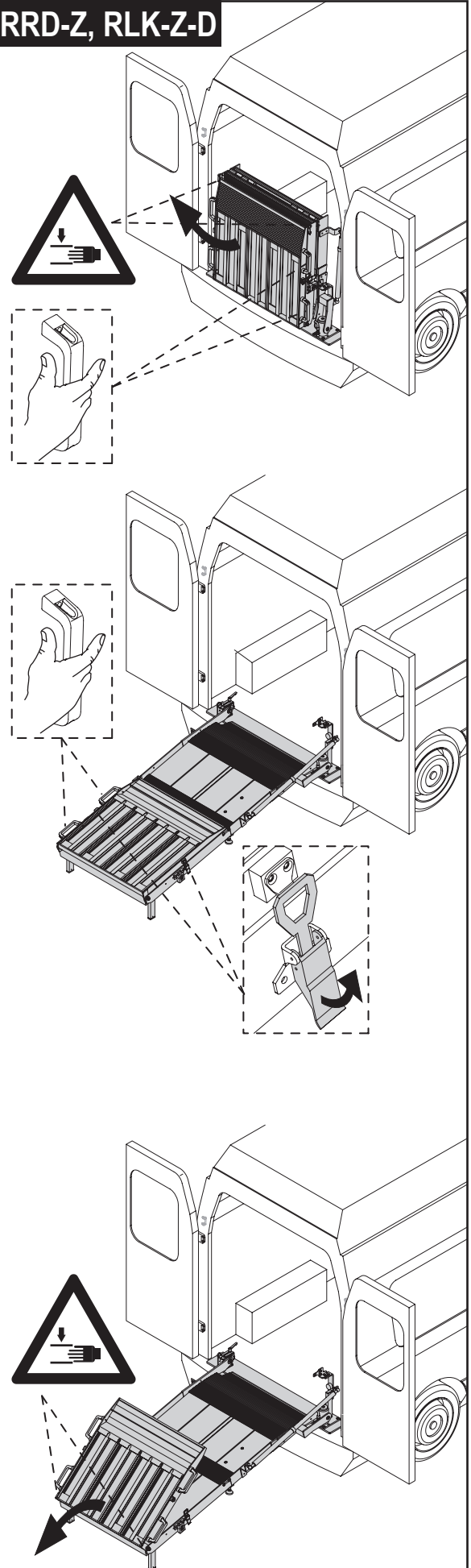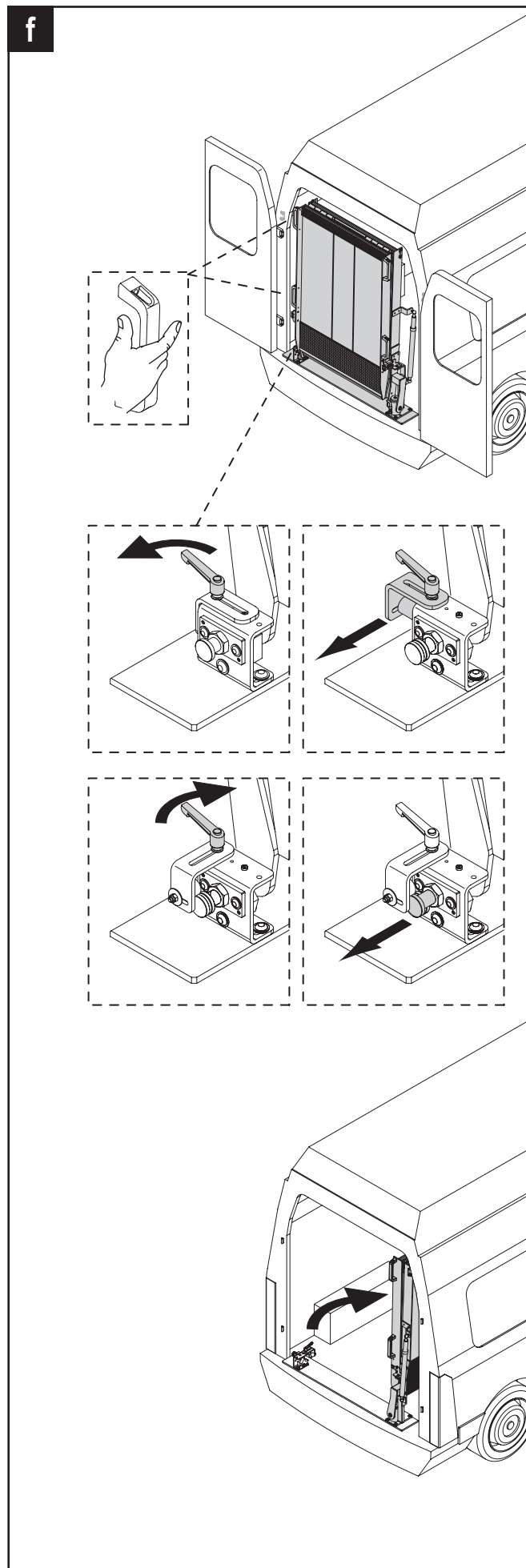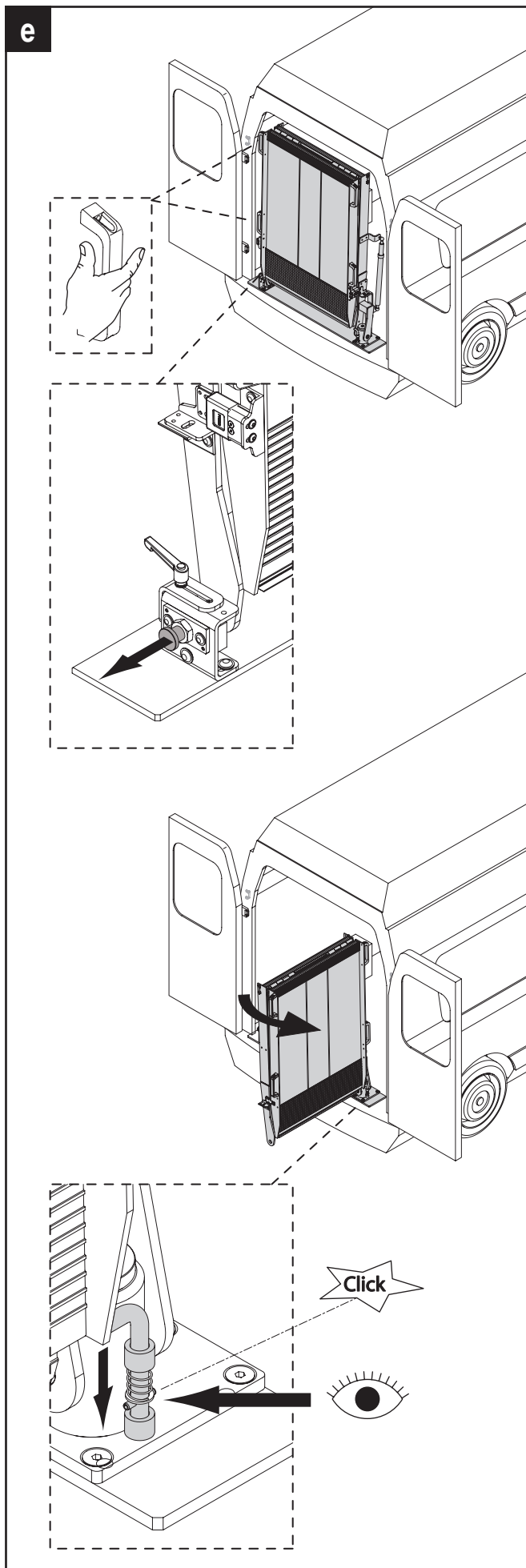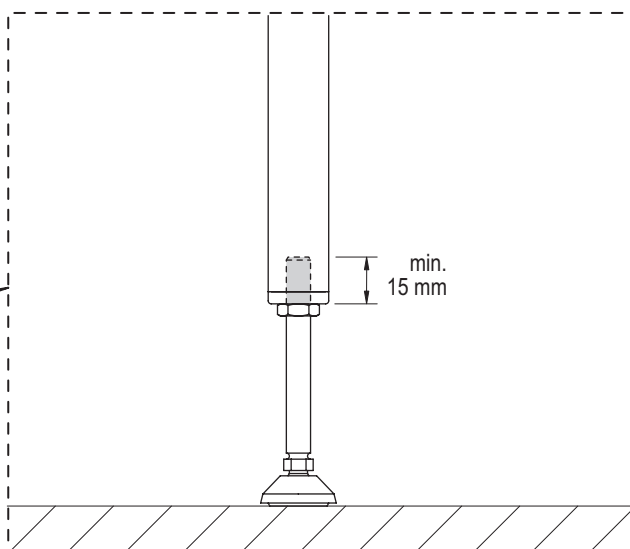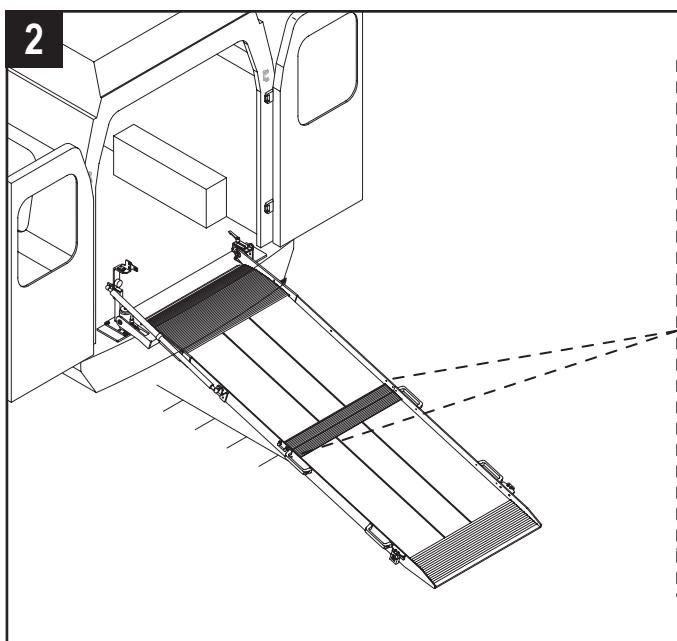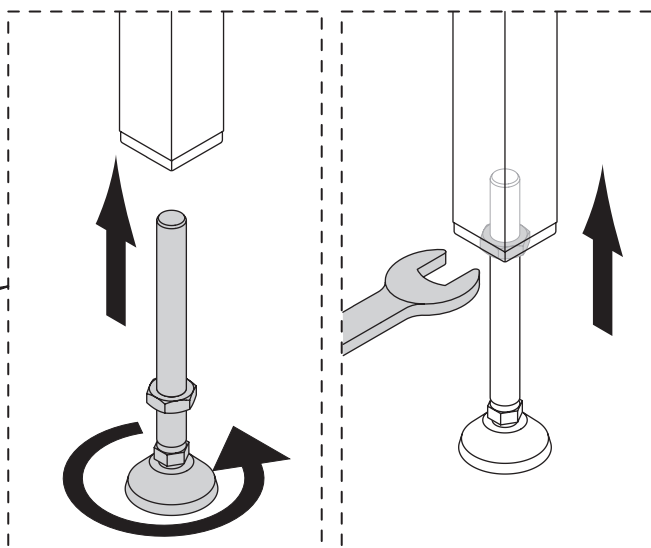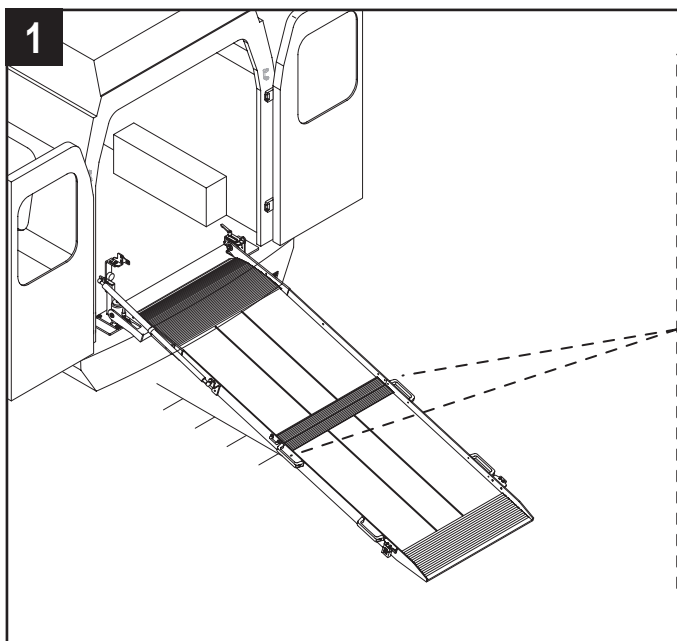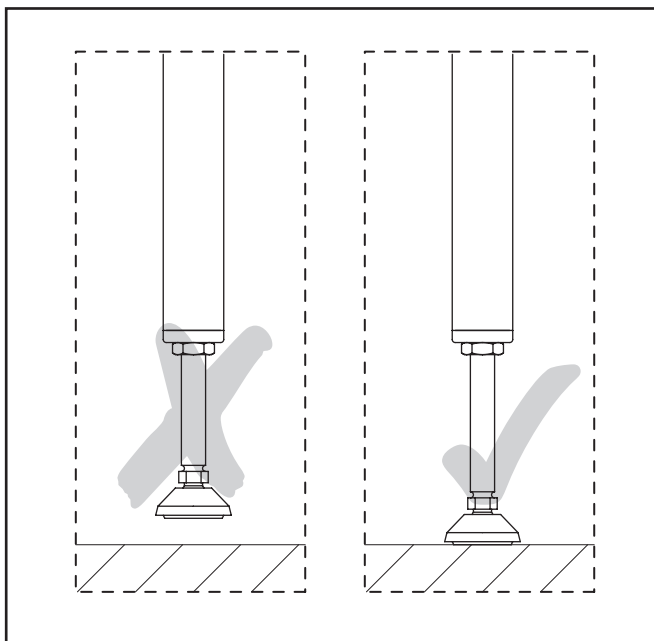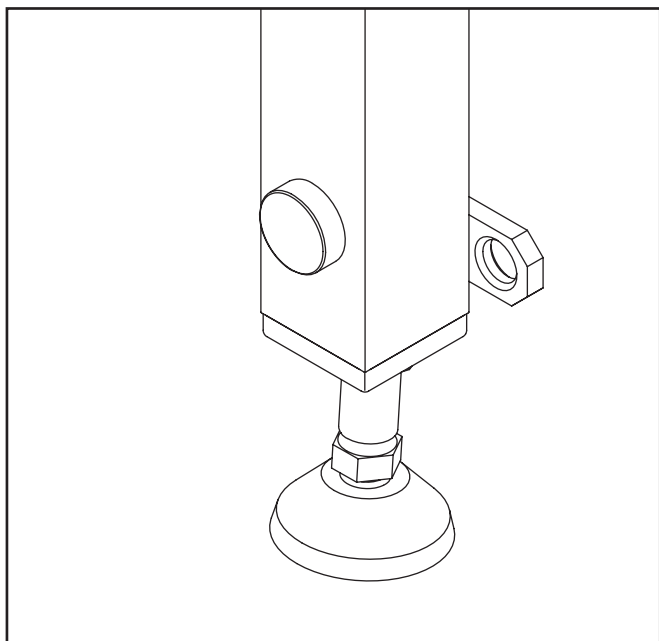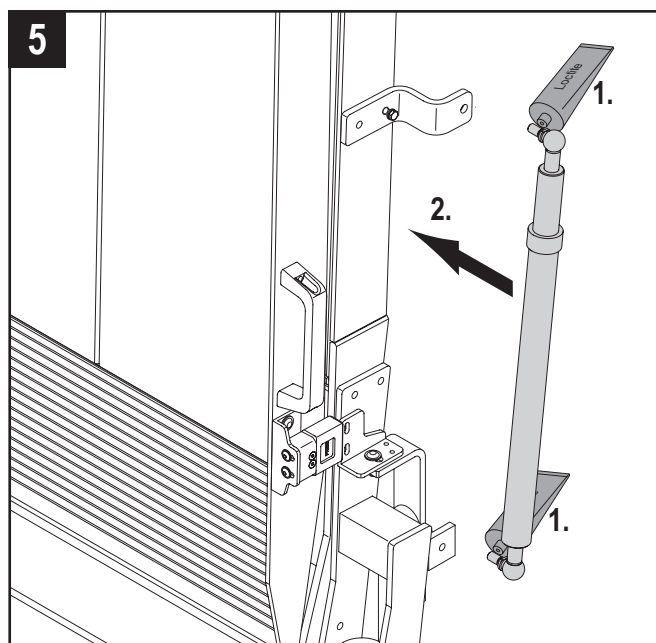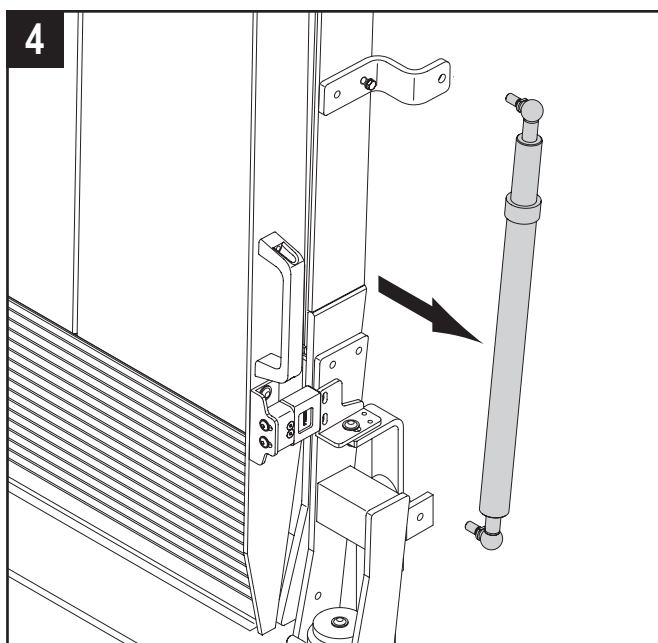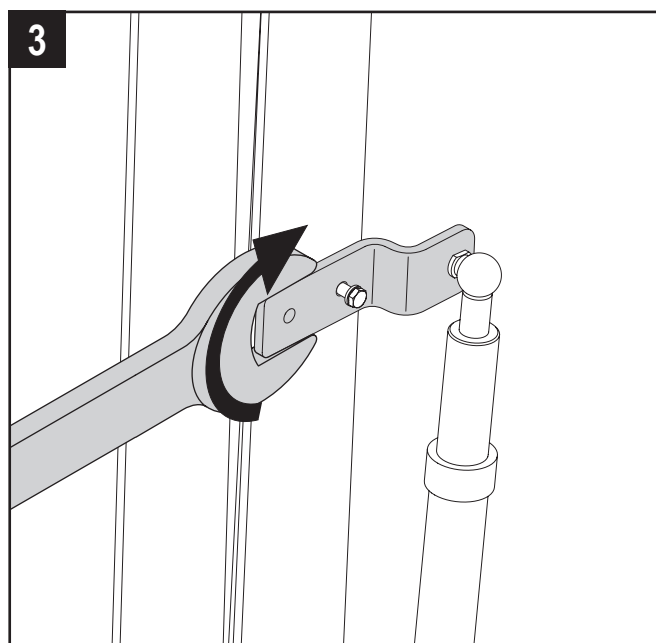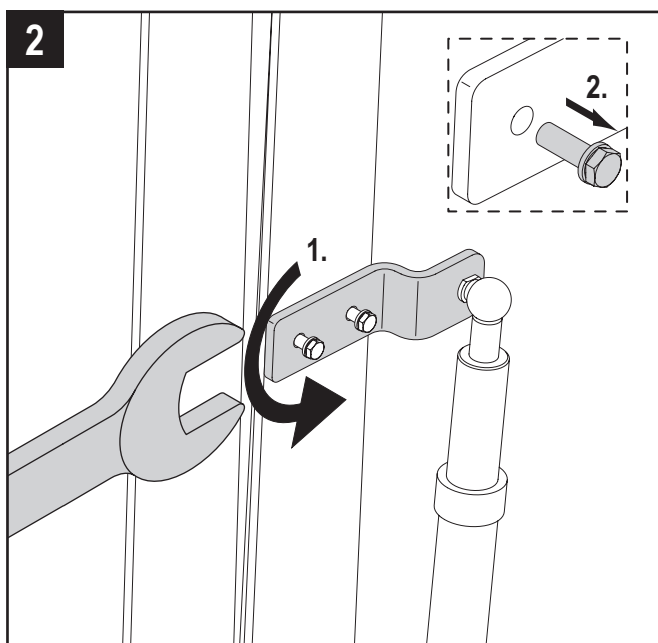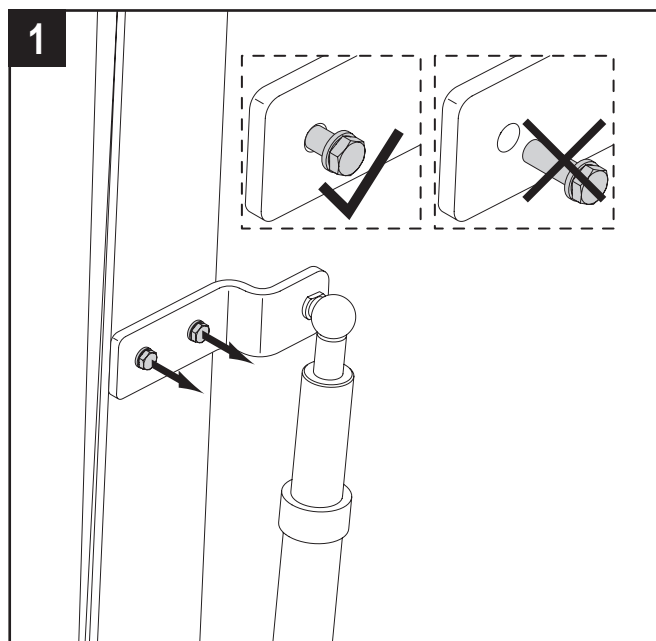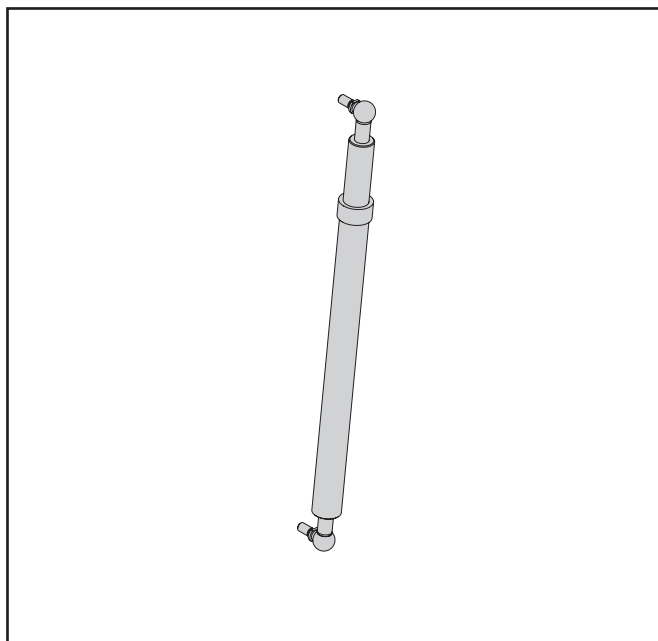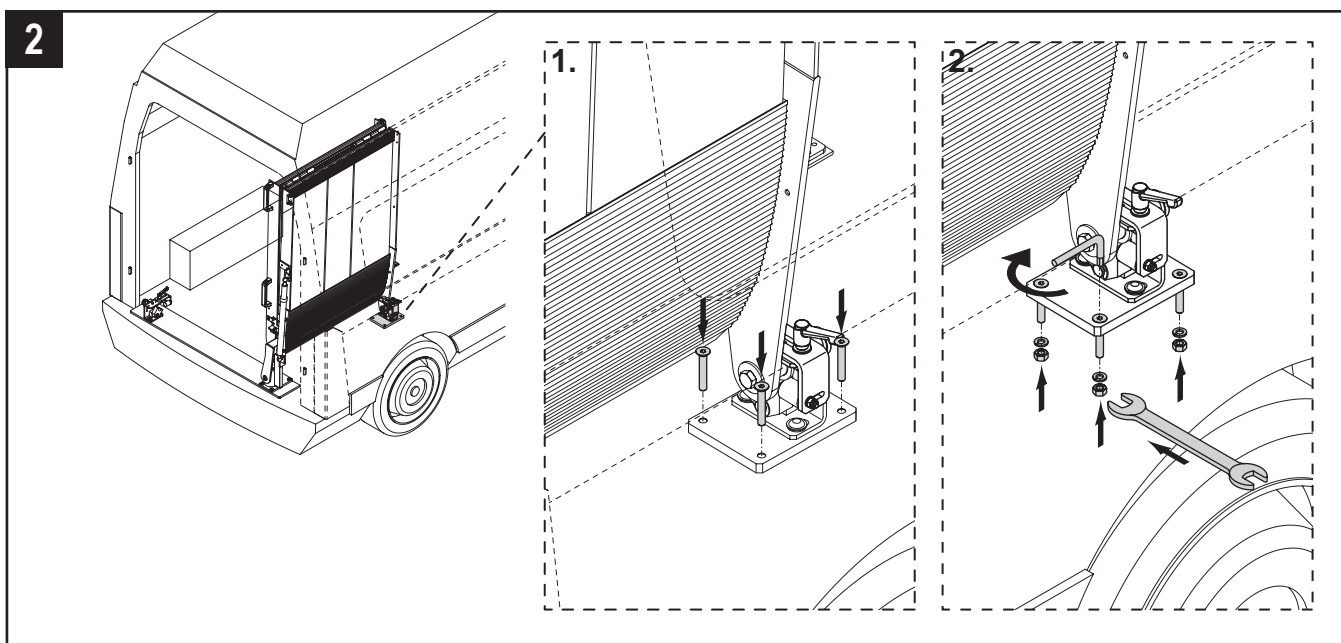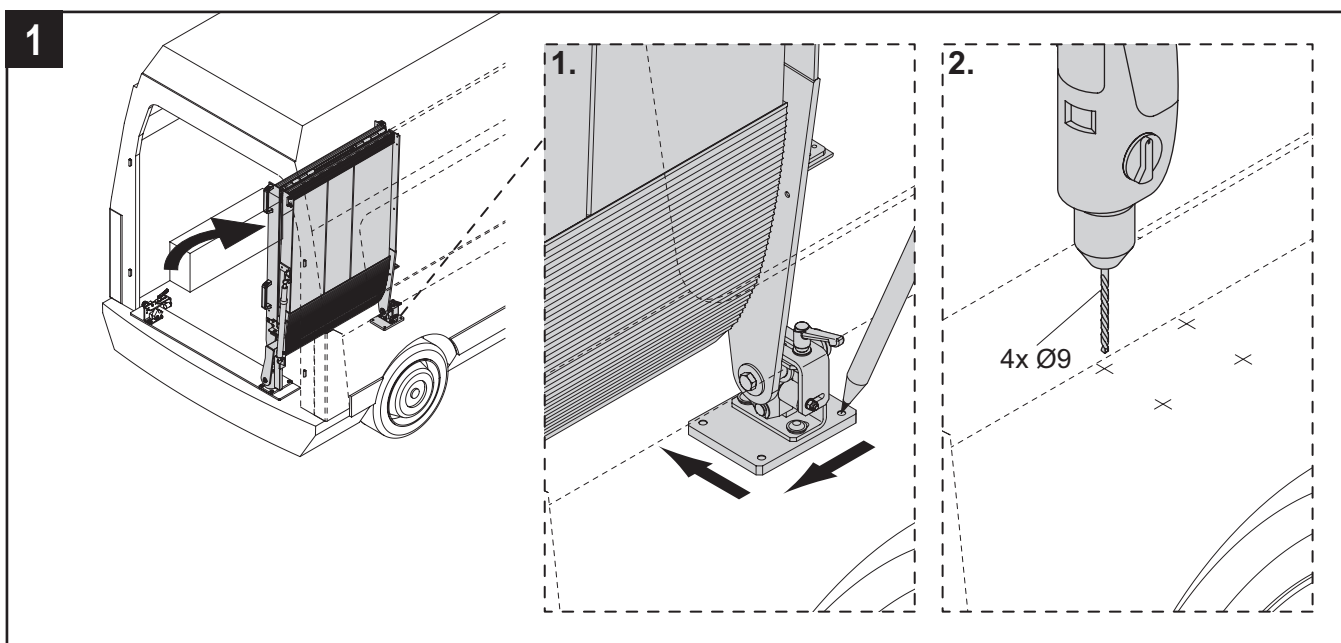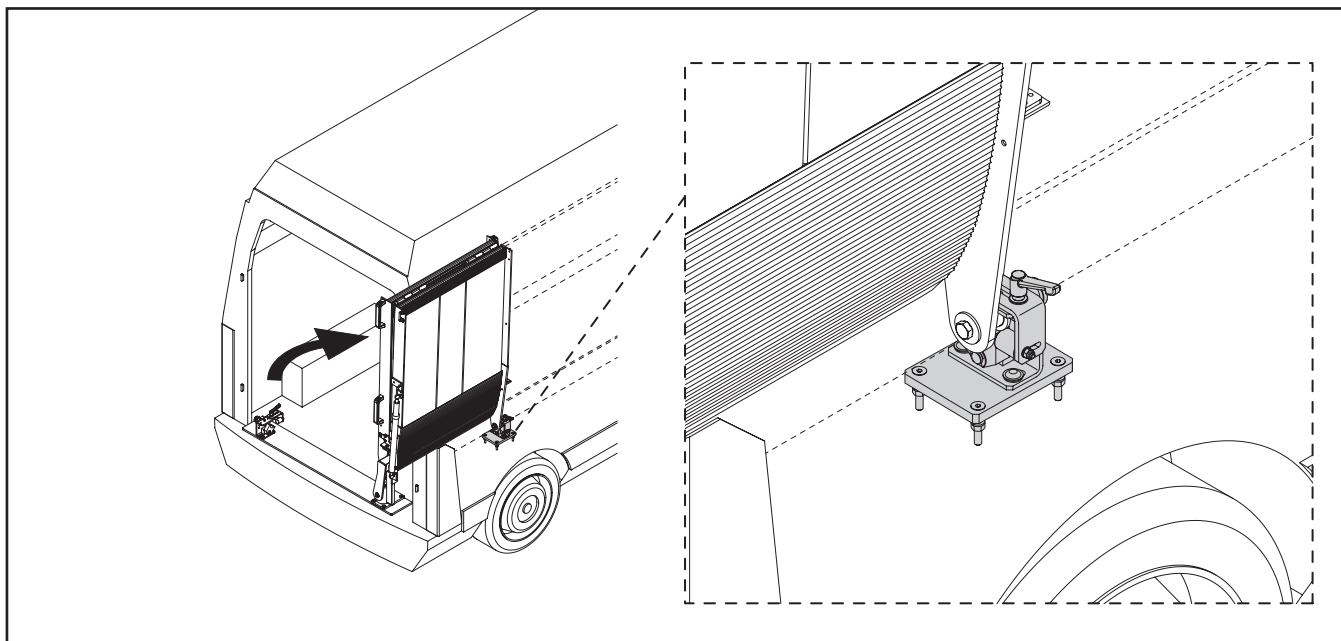D

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GB

Safety instructions

- Read the operating manual carefully. Only then may the product be utilised.
- Modifications to the product are subject to prior manufacturer approval.
- The product may only be used on stable terrain. The product must stand safely, i.e. it must not wobble.
- The product must be secured in such a manner that persons will not be endangered under normal traffic conditions.
- Loading and unloading of the vehicle must be carried out in such a manner that persons will not be endangered by objects falling or rolling down.
- The product when unfolded must be secured by the vehicle driver in such a manner that other road users (e.g. pedestrians, cyclists) will not be endangered.
- An incline >20% is not permissible in keeping with DIN 75078
- Follow the notices on the operation of electric wheelchairs
- Pneumatic spring under high pressure. Do not open! Do not heat above 80 °C.
- Impact or blows on the pneumatic spring will cause defects.

Check

Rule of thumb: an overloaded loading dock may not be used.

Before first commissioning

- Check all screw connections for proper seating
- Functional check
- Inspect pneumatic extension spring for damages.

Prior to each operation

- Visual check for any externally visible wear, tear and tear or damage (welding seams, drive surface)
- Completeness and full availability of all safety devices
- Inspect pneumatic extension spring for damages.

Monthly maintenance

- Check axial securing device of the hinge bolts for a firm seat.

Semi-annual maintenance

- Spray the hinges with a light machine oil.
- Visual check for any externally visible wear, tear and tear or damage (welding seams, drive surface)
- Functional check
- Check all screw connections for proper seating
- Clean any contamination from the running surface to ensure continuous „traction“.

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